

## AED Maintenance Check

**Daily, Monthly, Yearly**

School: \_\_\_\_\_

AED ID#: \_\_\_\_\_

*(ID # can be found on back of AED)*

Location of Unit: \_\_\_\_\_

\_\_\_\_\_ School Year

*Daily Maintenance: Check the STATUS INDICATOR to ensure that it is GREEN. When the indicator is GREEN, the Powerheart AED G3 is ready for a rescue. If the indicator is RED, refer to the Troubleshooting Table in the manual, located in the AED carrying case.*

### Daily Maintenance Check

Date	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31
Aug																															
Sept																															
Oct																															
Nov																															
Dec																															
Jan																															
Feb																															
Mar																															
Apr																															
May																															
Jun																															
Jul																															

*(Please initial each box)*

### Monthly Maintenance Check *(please initial)*

Aug _____	Feb _____
Sept _____	Mar _____
Oct _____	Apr _____
Nov _____	May _____
Dec _____	Jun _____
Jan _____	Jul _____

### Monthly Maintenance:

1. Open the AED lid.
2. Wait for the AED to indicate status.
3. Observe the change of the STATUS INDICATOR to RED. After less than 5 seconds, verify that the STATUS INDICATOR returns to GREEN.
4. Listen to the voice prompts.
5. Close the lid and confirm that STATUS INDICATOR remains GREEN, (for the Powerheart AED G3 only).

### Yearly Maintenance Check: Date \_\_\_\_\_ *(please initial)*

#### Yearly Maintenance:

- |   |  |
|---|--|
| <ol style="list-style-type: none"> <li>1. Open the AED lid.</li> <li>2. Remove the electrodes.</li> <li>3. Close the lid.</li> <li>4. Confirm that the STATUS INDICATOR turns RED.</li> <li>5. Open the lid and confirm that the Electrode Indicator is lit.</li> <li>6. Reconnect the electrodes and close the lid.</li> </ol> | <ol style="list-style-type: none"> <li>7. Make sure the expiration date is visible through the clear window of the lid.                     <ol style="list-style-type: none"> <li>a. check to make sure that the STATUS INDICATOR is GREEN.</li> </ol> </li> <li>8. Open the lid and confirm that no diagnostic indicators are lit.</li> <li>9. Check the expiration date of the electrodes.</li> <li>10. Check the electrode's packaging integrity.</li> <li>11. Close the lid.</li> </ol> |
|---|--|